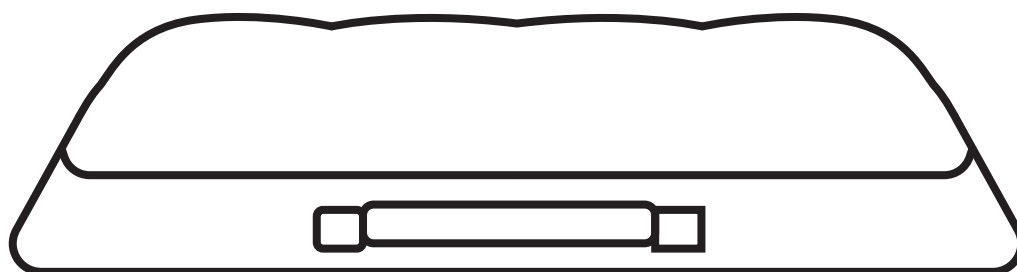
TheraMart

TM270

EN

INSTRUCTION MANUAL

TheraAir Cushion

PT

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Almofada TheraAir

ES

MANUAL DE INSTRUCCIONES

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[EN]

This instruction manual will present precautions and the safe use and operation of this cushion.

- Before using this cushion, carefully read this instruction manual to fully understand the safe and correct operational procedures.
- The correct operational procedures should not only be explained to the person using this cushion, but the caregiving staff must also be informed.
- After reading, keep this instruction manual properly stored for future reference.
- If you have any questions, please contact TheraMart directly.

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SUMMARY | SUMÁRIO | CONTENIDO

[EN]

1. Product Specifications	3
2. Maintenance	3
3. Intended Use	4
4. Important Safety Information	4
5. Cushion Instructions	4
6. Troubleshooting	5

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
[ENGLISH]

1. Product Specifications

Sizing (all measurements are approximate)	
W x D in	W x D cm
16" x 16"	41 x 41
16" x 18	41 x 46
17" x 17"	44 x 44
18" x 18"	46 x 46
20" x 18"	51 x 46
Cell Height Range: 2.75 in./ 7 cm without load	

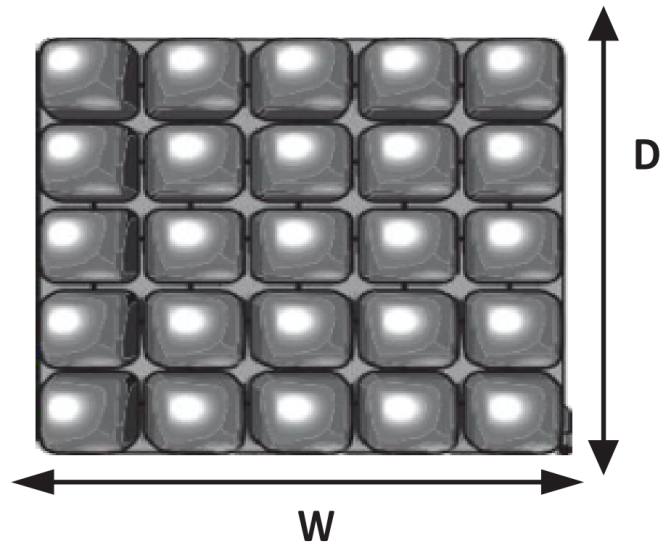2. Maintenance

⚠ Warnings:

- Clean the cushion regularly with neutral detergent or alcohol.
- Heating or steam sterilization is not available.
- When not using the cushion, store it in cool and dry place, keep away from the corrosive object.

Warranty

- The cushion is covered by 1-year warranty.
- The warranty does not apply to punctures, tears, burns, or misuse, nor does it apply to the removable cover.
- The manufacturer reserves the right to change the parameters without prior notice.

3. Intended Use

The cushion is an adjustable, air-filled, segmented-design wheelchair support surface and is intended to conform to an individual's seated shape to provide skin/soft tissue protection. There is a weight limit of 315lbs.(143kg), and the cushion must be properly sized to the individual.

4. Important Safety Information

Cautions:

- Keep the cushion away from sharp objects.
- Changes in altitude may require adjustment to the cushion.
- A cushion that has been in temperatures < 32°F/0°C may exhibit unusual stiffness. See "Troubleshooting".
- DO NOT use the inflation valve as a handle for carrying or for pulling the cushion. Carry your cushioning device by the cushion base or by the carrying handle on the cover.
- Using a pump, cover, or repair kit, other than those compatible with your product may void the product warranty.
- DO NOT modify the cushion nor any components. Doing so may result in damage to the product and will void the warranty.
- Periodically, check the cover and components for damage and replace if necessary. See " Troubleshooting".

5. Cushion Instructions

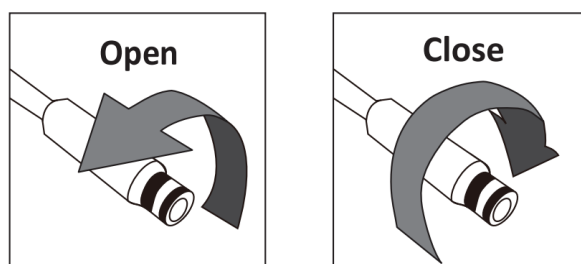
Cushion Inflation:

1. Overinflate the cushion:
 - a. Slide the hand pump nozzle over the inflation valve. Open the inflation valve.
 - b. Inflate the cushion until all of the air cells feel firm.
 - c. Close the inflation valve and remove the hand pump.

Cover Placement:

2. Place the cushion inside the cover:
 - a. Unfold the cover.
 - b. Insert the cushion.
 - c. Close the zipper.

Note: When covered, the cushion should lie flat on the chair (non-skid material on the bottom).


6. Troubleshooting

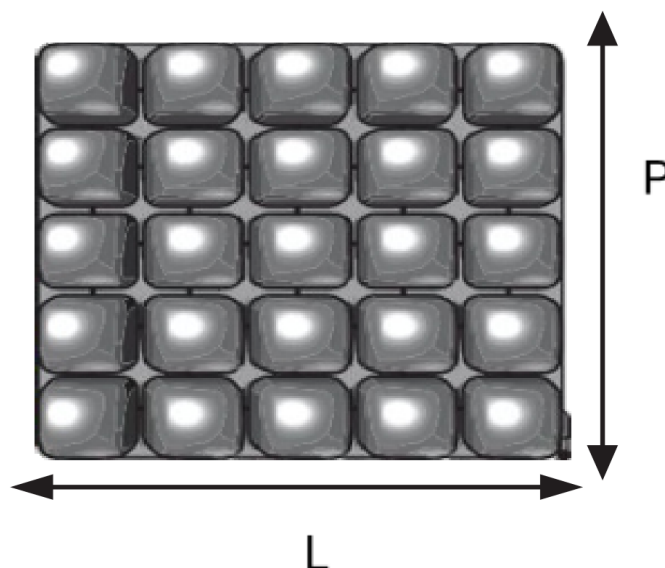For additional assistance, contact your equipment provider, distributor, or Customer Support.	
Not Holding Air	Inflate the cushion. Inspect the inflation valve and hose for damage. Confirm that the inflation valve is completely closed. Look for holes in the cushion. If very small holes or no holes are visible, use the repair kit provided with your product. For inflation valve or hose damage, or for large holes or leaks in the cushion, please contact customer support.
Not Able To Inflate or Deflate the Cushion	Confirm that the inflation valve is open. Inspect the inflation valve and hose for damage.
Uncomfortable or Unstable	Confirm that: - The inflation valve is in the front, on the left side of the seated individual. - The cushion is not under inflated or over inflated. - The cushion cells and the top of the cover are facing up. - The cushion is centered beneath the individual. - The cushion is properly sized to the individual and the wheelchair. Allow the individual at least one hour to adjust to the cushion
The Cushion is Stiff	If the cushion has been in temperatures lower than 32°F (0°C) and exhibits unusual stiffness, allow the cushion to warm to room temperature.
Component is Damaged	Contact Customer Support for replacement.

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