TheraLift Slim

SILLÓN / POLTRONA / CHAIR

ES Manual de instrucciones **PT** Manual de instruções **EN** Instruction manual

TM3020 • TM3021

TheraMart

TheraLift Slim

SILLÓN / POLTRONA / CHAIR

ESPAÑOL

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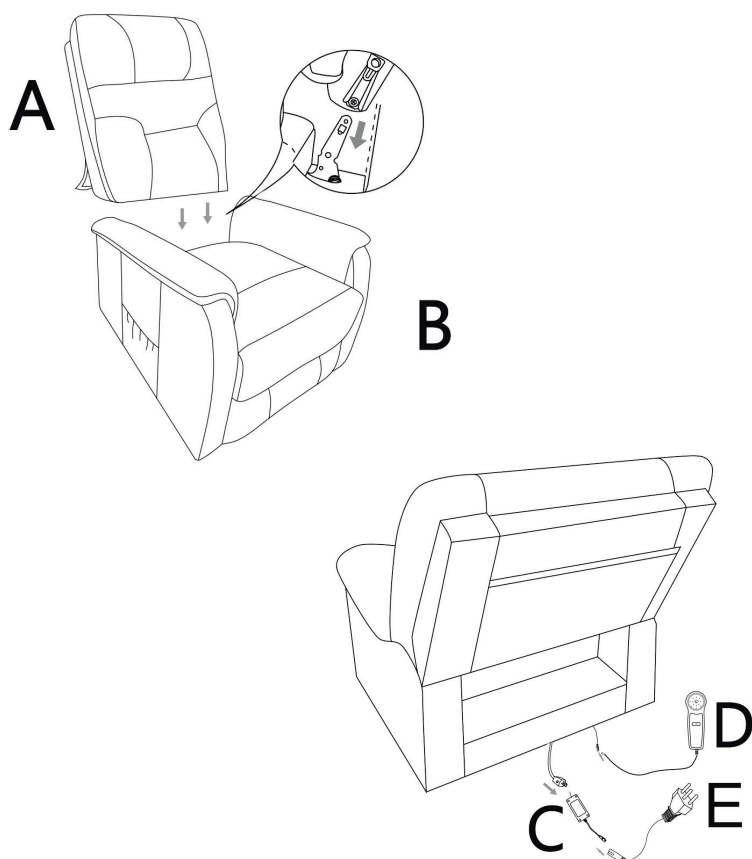
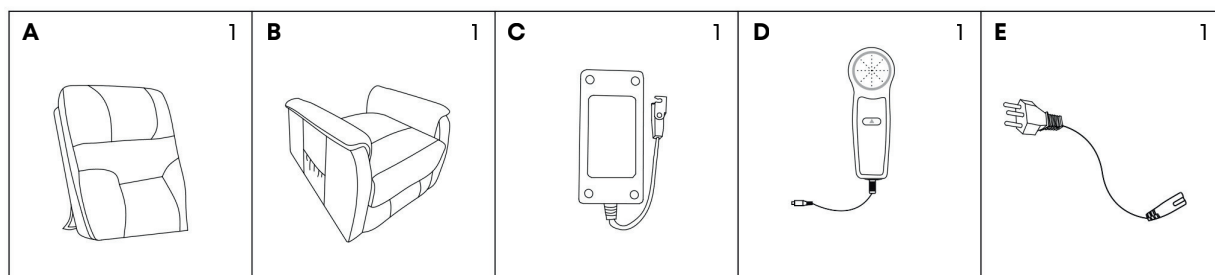
Assembly instructions

Read this manual carefully before assembly. Inspect all cartons to confirm that no parts came loose during transportation. Remove all components from the packaging and place them on a soft surface such as carpet, a blanket, or a furniture pad. Identify and inspect all parts and accessories before assembly.


ASSEMBLY STEPS:

1. Install the backrest (A) onto the chair base (B).
2. Connect the controller (D) to the corresponding cable of the chair.
3. Connect the power adapter (C) to the power cable with plug (E). Plug it into the wall outlet to complete the assembly.

WARNING

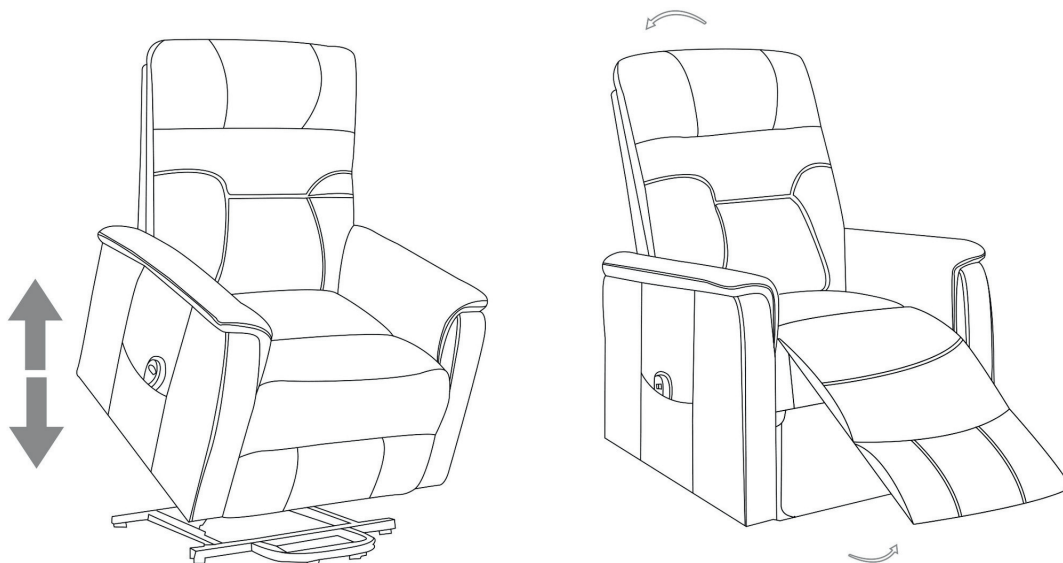- Do not stand in this chair, do not use this chair as a stop ladder
- Use this chair only for seating one person at a time
- Do not use this chair unless all bolts, screws and knobs are firmly secured
- Check all bolts, screws and knobs on a regular basis
- This chair should not be used with damaged or missing parts
- Contact and authorized service provider to repair or replace damaged or missing parts
- Failure to follow these warnings could result in serious injury

1. Up arrow (press and hold)

Activates the **Power Lift** mode, tilting the seat forward to help you stand up almost completely without putting pressure on your knees or hips.

When held down, the footrest closes and the chair begins to rise smoothly. **Movement stops immediately when the button is released.**

2. Down arrow (press and hold)

Allows you to reach a deep relaxation position (up to 135°), ideal for improving leg circulation and relieving tension in the lower back area.

When held down, the backrest begins to recline and the footrest rises in sync. **Movement stops immediately when the button is released.**

3. USB-C and USB-A port

Located at the top of the controller, compatible with USB type-C and type-A charging interfaces.

They allow you to keep your mobile devices (phones or tablets) charged while you rest, preventing you from having to get up unnecessarily.

TM3020
NOIR

TM3021
BROWN

Product Data

Description

Electric lift chair

Up to 3 positions:

- 135° recline angle
- Lift
- Upright chair position

The armchair can remain in intermediate positions. The position is adjustable according to the user's needs and requirements.

Product parts

- Padded backrest
- Padded armrests
- Padded seat
- Side pocket
- Chair position control
- USB type-C and type-A ports on the controller

Materials

- **Product:** iron frame, motor, and wooden frame
- **Filling:** cotton foam and original foam
- **Cover material:** PU
- **Base fabric:** 40% TC + 60% PU surface
- **TC:** 12.5% cotton + 87.5% polyester
- **PU:** 70% polyurethane + 29% filler + 1% colorant

Maximum supported weight

150 kg

Product dimensions

- **Overall dimensions:** 76.5 x 95 x 103 cm
- **Seat:** 45 cm width x 54.5 cm depth x 48.5 cm height
- **Armrest height:** 60 cm
- **Backrest height:** 68 cm
- **Reclined backrest height:** 85 cm
- **Total depth when fully extended:** 154 cm
- **Open footrest height:** 52 cm

Angle of inclination

Closed: 108° / **Open:** 135°

Storage Data

Net weight | Product

35 kg

Shipping box dimensions

76 x 76 x 65 cm

Shipping box weight

39 kg

Units per shipping box

1

Cleaning instructions

- **Permitted cleaning products:** water, mild soap and water, diluted 70% isopropyl alcohol, certified special disinfectant wipes.
- **Not recommended or prohibited:** strong corrosive chemicals, abrasive cleaners, strong organic solvents, high-pressure steam cleaners, or direct rinsing with water.

Preventive maintenance

- **Daily cleaning:** clean with a soft, dry or slightly damp cloth.
- **Stains:** use a neutral detergent or a special cleaner and wipe gently.
- **Damage prevention:** avoid direct sunlight, humidity, and high temperatures, and use conditioner every 3–6 months.

Storage conditions

- Dry, ventilated, cool indoor place.

Technical Specifications

System

- Electric.
- Only in case of electrical failure or power outage, the chair will automatically return to its original position.

Nominal output voltage

DC29.0V 2.0A 58.0W

Input voltage range

100–240V~50/60Hz 1.5A

Warranty

1-year warranty.
Damage caused by the user is not covered.

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